

Manman Plaster Cutter

P 5 A

Manman
Plaster Cutter

"Safety is Our Power"

To improve the corrosion resistance & durability the blades are PTFE Coated. Blades are having clear fit & snap on mounting hole. It can be adjusted in six positions by rotation for prolonged use.

Di 66 L / Di 67 C / Di 68
Di 69 * C / Di 70 *

*not the possible space against any other equipment about the size it steps will be concerned properly, if required.

Unique Depth Control Gauge

Removing & Fitting the Saw Blades

First, put the Pin Spanner in the hole on the plaster cutter shaft. Have a clear grip on it.

Then use regular Nut Spanner for releasing or locking the Dorse Nut.

After use, Plaster Cutter should be wiped with a dry cloth & stored properly in the container.

Brush & Duster is provided.

Every plaster cutter is provided with a bag to accommodate the Plaster Cutter, Supply Cable, Saw Blades, Pin Spanner, Spanner, Brush, Duster & Preventive maintenance card.

See that the cutter is not coming in contact with the electric cable. This may harm the PVC Supply Cable.

Comfortable grip is provided for better balancing and cushioning effect, it is achieved with concrete handles. It also provides additional support while handling the cutter.

Holes are provided for ventilation.

Plaster Cutter is to be connected with the main with push on type connector cable.

On/Off Switch

In case of any problem please check the Fuse (amp, single pole).