


SNELL
SURGiZYME

SNELL SURGiZYME

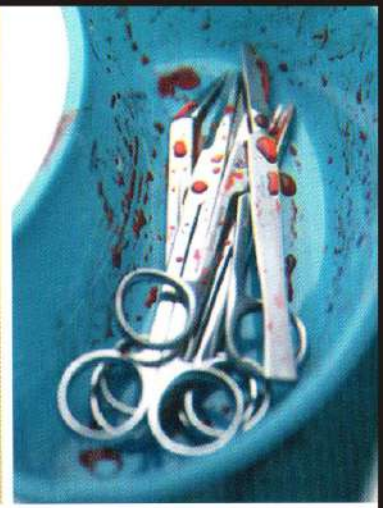

ADVANTAGE.


- Provides quick and thorough cleaning.
- Facilitates effective cleaning and rapid turn around
- Immediately begins to break down Blood, Tissue, Mucous and other Protein rich body fluids
- No other cleaning ingredients required.

RECOMMENDED USES

- Medical /surgical instruments
- Respiratory therapy equipment.
- Evacuation system.
- Stainless steel, Tungsten carbide, Aluminum, Brass, Plastic materials.
- Delicate micro surgical instruments
- Flexible and Rigid scopes.

DIRECTION FOR USE

Light Biological Contamination	5 ml/ltr. of water Soak Time 2-5 min.
Moderate Biological Contamination	7.5 ml/ltr. of water Soak Time 8-10 min.
Heavy Biological Contamination	10 ml/ltr. of water Soak Time 10 min.
Biological Contamination have Dried	20 ml/ltr. of water Soak Time 20 min.

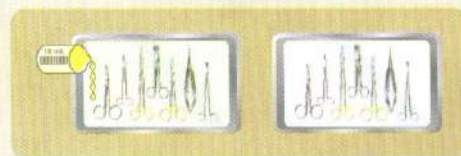

Available in • 100 ml. • 1 Ltr. • 5 Ltr. Pack

SPECIFICATION :

Appearance	: Liquid
Colour	: Floracent Yellow
Detergency	: Excellent
Foaming	: Low foam
Solubility	: Complete with water
Rinsing Properties	: Excellent
Corrosion Factor	: Non - corrosive
PH	: Neutral
Biodegradable	: All ingredients are biodegradable

Powerful pre soak surgical instrument and scope cleaner

SNELL SURGiZYME is a unique combination of enzymes which effectively remove Blood, Tissue, Mucous and other protein rich body fluids, from Stainless Steel, Tungstencarbide, Aluminum, Brass, Glass, Plastic Materials, Delicate micro surgical instruments, Flexible and rigid scopes.

SNELL SURGiZYME is use as pre soak surgical instrument and scope cleaner

SNELL SURGiZYME immediately works to break down Blood, Fat and Protein and saves reprocessing time by initiating the pre - cleaning process. For most application there is no need to add further cleaning ingredients for pre soaking or manual cleaning.

SNELL SURGiZYME saves time, lower inventory levels, cuts the cost of consumables and improves turn around time.

YOUR
SURGICAL
INSTRUMENT
WILL LOOK
NEWER
AND
PERFORM
LONGER



* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western Surgical

WESTERN SURGICAL

8

www.westernsurgical.in
Email : westernsurgical@gmail.com
Ph. : +91-281-2232871 / 2243159

© Copyright Reserved by : Western Surgical - Rajkot.