




A - WITHOUT TIMER



B - WITH TIMER

FUMIGATION MACHINE

USER MANUAL - 2019

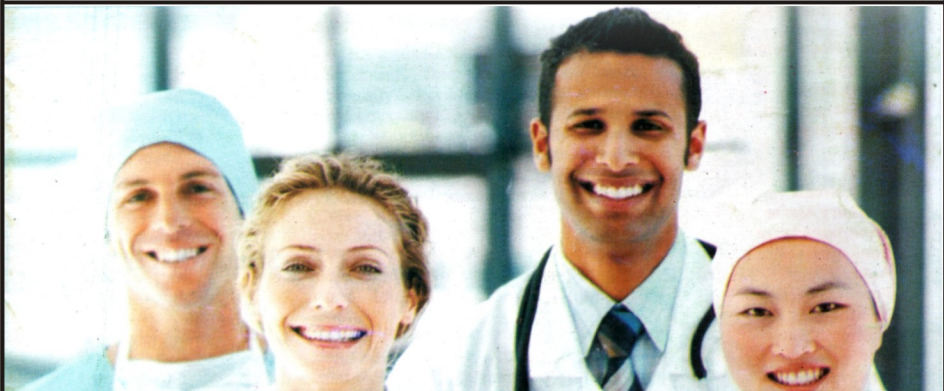

WESTERN SURGICAL

Western Surgical AN ISO 9001 : 2008 & ISO 13485 : 2015 AC 2012 CERTIFIED COMPANY

www.westernsurgical.in Email : westernsurgical@gmail.com

Email : info@westernsurgical.in Ph. : +91-281-2232871 / 2243159

HOW TO OPERATE FUMIGATION MACHINE

1. FOR 350 SQ. FT. OT TAKE 1000 ML (1 LIT) WATER IN TANK AND ADD SOLUTION IN THAT. (CHANGE QTY OF WATER AND SOLN AS PER YOUR AREA AND RECOMMENDATION. OF MICROBIOLOGIST)
2. KEEP FUMIGATION MACHINE ON FLOOR.
3. KEEP MACHINE NEAR OT DOOR SO AFTER FUMIGATION YOU CAN EASILY SWITCH OF MACHINE.
4. ADJUST METERING VALVE (SMALL ROUND KNOB ON TANK) ON LOW COMPULSORY SO IT WILL GIVE MINIMUM MICRON PARTICAL FOR BETTER FOG AND PENITRATION.
5. ADJUST MACHINE DIAGONALY OPPOSITE TO ANOTHER CORNER WITH APPROX. 45 DEGREE TILT TOWARDS SKY. MAKE PATH CLEAR FOR FOG TO FLOAT IN AIR FOR LONGER PERIOD.
6. START MACHINE FOR 15-20 MUNITE FOR 1-LIT WATER AND 30-35 MIN FOR 2-LIT WATER.
7. PUT OFF MACHINE AFTER COMPLETION OF TIME AND KEEP OT CLOSE OR SEAL FOR MINIMUM 1-2 HOUR.
8. WASH FILTER UNDER RUNNING WATER AFTER EVERY FOGGING. WHICH IS ON THE BACK SIDE OF MACHINE AND KEEP IT BACK ON PLACE AFTER WASHING.
9. EMPTY THE MACHINE TANK AND USE BALANCE SOLUTION FOR MOPPING. ADD SOME PLANE WATER IN TANK AND RUN MACHINE FOR 2 MIN. IT WILL CLEAN INTERNALLY.
10. DAILY MOPPING OF OT IS COMPULSORY WITH SOLUTION.
11. PLEASE COVER ALL ELECTRO MEDICOS, DIGITAL & WATER RESISTANCE EQUIPMENTS (C ARM, MONITOR, MICROSCOPE ETC.) WITH CLOTH OR PLASTIC SHEET
12. DO NOT RUN MACHINE CONTINUOUSLY MORE THAN 35 MIN. AND TOTAL ONE HOUR IN A DAY; KEEP 4 TO 6 HOURS OF INTERVAL IN BETWEEN TWO RUNNING.
13. DO NOT OPERATE MACHINE IN PRESENCE OF PATIENT AND DO FUMIGATION IN VACCANT ROOM ONLY.

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



WESTERN SURGICAL

Western Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

①

© Copyright Reserved by : Western Surgical - Rajkot.

7.3 WARRANTY CERTIFICATE

western surgical product is warranted to be free from electrical and mechanical defects in material and workmanship, under normal use. The western surgical product must not have been moved from the site of original installation. A new or remanufactured part to replace the defective part will be provided without charge for the part itself, through an authorized western surgical service dealer. The replacement part assumes the unused portion of the warranty.

WESTERN SURGICAL WILL NOT BE RESPONSIBLE FOR:

1. Damage or repairs required as a consequence of faulty installation or application by others.
2. Damage or repairs needed as a consequence of any misapplication, negligent handling, improper servicing, unauthorized alteration, or improper operations.
3. Failure to start due to voltage conditions, blown fuses, open circuit breakers or other damages due to the inadequacy or interruption of electrical service.
4. Damage as a result of floods, winds, fires, lightning, accidents, corrosive atmosphere, or other conditions beyond the control of western surgical
5. Parts not supplied or designated by western surgical.

Date of Installation	Installed By
Model No.	Serial No.
Warranty Period	
Name of Doctor & Address :	

Customer Sign. With Stamps

Marketed By: Western Surgical
Sign. With Stamps

No Claim Warranty :

- 1) Any Defect Throughtout Power Supply
- 2) Any Physical Damage

3) Under Warranty Standby Unit Not Provide

4) Under Warranty When Company Send Parts or Machine we Imdiat send to Buyer.

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



WESTERN SURGICAL

Western Surgical

Dt. : 31-7-2019


www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

© Copyright Reserved by : Western Surgical - Rajkot.

OUR OTHER PRODUCTS RANGE

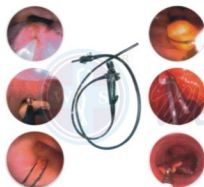

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.

WESTERN SURGICAL

Western Surgical

AN ISO 9001 : 2008 & ISO 13485 : 2015 AC 2012 CERTIFIED COMPANY

"Happy Home", 1st Floor, 23, Karanpara,
B/h. S.T. Bus Stand, Kanak Road,
Rajkot-360 001. Ph. : +91 281-2232871/2243159
Mo. : +91 98250 69069 - +91 98250 45997

Email : westernsurgical@gmail.com

Email : info@westernsurgical.in

* www.westernsurgical.in *

© Copyright Reserved by : Western Surgical - Rajkot.

