*Western Surgical*®

# Hygienic Solution

## USER MANUAL

[www.westernsurgical.in](http://www.westernsurgical.in)  
Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)  
Email : [info@westernsurgical.in](mailto:info@westernsurgical.in)  
Ph. : +91-281-2232871 / 2243159

## INSTRUCTIONS FOR USE OF NEEDLE BURNER

1. Insert power plug in switch board and switch on.
2. Put on the switch which is on the back-side of machine. A green Led will be indicated.
3. Insert needle in hole marked 'BURNER' and press. Needle will start melting, continue pressing till the needle has fully melted.
4. To reuse syringe remove the melted needle, insert the syringe in the broad slot marked "PULLER" and push it in to the narrow slot, then hold the part between needle and syringe. Tilt or pull syringe up so that the needle separates and falls into waste box kept below.
5. If syringe is not to be reused, put it in the slot marked "CUTTER" and cut it. That is after the needle has melted, put syringe in the "CUTTER" hole beside it till the needle holder is pushed in. Then turned the cutter handle forwards you. The syringe will get cut from the needle holder. To press handle put your finger on top of handle and thumb on the front part of the machine - syringe will cut easily.
6. Pull the knob installed in the front of the machine. So that the drawer comes out And any waste from burnt needle and cut syringe pieces can be emptied from the drawer (Time to Time) when needed. Drawer should be put back in place.
7. If machine is not to be used for long period keep the switch 'OFF'.
8. If machine does not start even after power switch is put "ON" turn the red button behind i.e. "FUSE and open. If the fuse has blown then put in a new fuse of 2 Amp slow blow type to restart.

\* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.

*Western Surgical*

[www.westernsurgical.in](http://www.westernsurgical.in)

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\* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.

Western Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

Date of Installation	Installed By
Model No.	Serial No.
Warranty Period	
Name of Doctor & Address :	

Customer Sign. With Stamps

Marketed By: Western Surgical  
Sign. With Stamps

**No Claim Warranty :**

**1) Any Defect Throught Power Supply**

**2) Any Physical Damage**

3) Under Warranty Standby Unit Not Provide

4) Under Warranty When Company Send Parts or Machine we Imidiat send to Buyer.

\* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.

**Western Surgical®**

"Happy Home", 2nd Floor, 23, Karanpara,  
B/h. S.T. Bus Stand, Kanak Road,  
Rajkot-360 001. Ph. : 0281-2232871/2243159  
Mo. : 98250 69069 - 98250 45997  
Email : westernsurgical@gmail.com  
Email : info@westernsurgical.in

\* [www.westernsurgical.in](http://www.westernsurgical.in) \*