



Western Surgical®

सेन्ट्रल लाईन सक्शननो
उपयोग करवानी रीत

USER'S MANUAL



www.westernsurgical.in

Email : westernsurgical@gmail.com

Email : info@westernsurgical.in

Ph. : +91-281-2232871 / 2243159

॥ श्री गणेशाय नमः ॥

सेन्ट्रल लाईन सफ़्शननो उपयोग करपानी रीत



* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

- ❧ સેન્ટ્રલ લાઈન સકશન દરેક બેડ ઉપર સેન્ટ્રલ લાઈન સકશનની પાઈપ લાઈનનું કનેક્શન કરેલ છે તથા ઓપરેશન થીચેટરમાં પણ કરેલ છે. તેની બાજુમાં બોટલ સ્ટેન્ડ મુકેલ છે.
- ❧ જે બેડ ઉપર સેન્ટ્રલ સકશનની જરૂર પડે ત્યારે ઉપરની મેટલ કનેક્શન વાળી નોઝલને પાઈપ લાઈનનાં વાલમાં ફીટ (નોઝલ ઉપરની તરફ પ્રેસ કરવાથી સકશન યુનિટ ચાલુ થઈ જશે.)
- ❧ સકશન કર્યા પછી નોઝલને વાલ માંથી છુટી કરી દેવી.
- ❧ વાલને ડાબી બાજુથી જમણી બાજુ ફેરવવાથી નોઝલ નીકળી જશે.
- ❧ સકશનની જરૂરીયાત ન હોય ત્યારે બોટલમાંથી મેટલ નોઝલની પાઈપને ખાસ છુટી કરવી અને તેનું ધ્યાન રાખવું જરૂરી છે. નહીંતર સકશન કરવાનું બંધ કરી દેશે.
- ❧ જરૂરીયાત પ્રમાણે જ સકશન બોટલની નોઝલને પાઈપલાઈન જોડે જોઈન્ટ કરો.
- ❧ ઉપરની વિગત પ્રમાણે સેન્ટ્રલ લાઈન સકશન વાપરવાથી સકશન ઘણું જ સાફ ચાલશે.

* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western Surgical®

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

Date of Installation	Installed By
Model No.	Serial No.
Warranty Period	
Name of Doctor & Address :	

Customer Sign. With Stamps

Marketed By: Western Surgical
Sign. With Stamps

No Claim Warranty :

1) Any Defect Throught Power Supply

2) Any Physical Damage

3) Under Warranty Standby Unit Not Provide

4) Under Warranty When Company Send Parts or Machine we Imdiat send to Buyer.

* Carrying Warranty at Rajkot only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.

Western Surgical®

"Happy Home", 2nd Floor, 23, Karanpara,
B/h. S.T. Bus Stand, Kanak Road,
Rajkot-360 001. Ph. : 0281-2232871/2243159
Mo. : 98250 69069 - 98250 45997
Email : westernsurgical@gmail.com
Email : info@westernsurgical.in

* www.westernsurgical.in *

