



BABY SCALE

USER MANUAL

 **Western Surgical**
AN ISO 9001 : 2008 CERTIFIED COMPANY

www.westernsurgical.in Email : westernsurgical@gmail.com
Email : info@westernsurgical.in Ph. : +91-281-2232871 // 2243159

INSTRUCTIONS MANUAL

Dear Customer,

We Thank you for buying the KIDLEE BABY SCALE.

Precise and easy to use, the digital Baby Scale allows you to weigh your baby with a precision of 5 gram accuracy. The zero function allows you to weigh the baby using a soft cloth or towel. The scale come with a useful weight-hold function that allows a steady weight to be displayed despite the child wriggling about. The Baby Scale has been designed to help you monitor the growth of your baby, accurately and reliably. This Scale can used in Medical Purpose for Urine measurement, Pre mature Baby's Growth & During Fluid loss in Baby.

INDEX

1. BEFORE USE
2. TECHNICAL CHARACTERISTICS
3. KEYS FUNCTIONS
4. NORMAL WEIGHING FUNCTIONS
5. WEIGHING FUNCTION WITH A TOWEL
6. UNIT SELECTOR FUNCTION
7. CARE & ATTENTION
8. LIMITED WARRANTY
9. GROWTH CHART
10. WARRANTEE

1. BEFORE USE :

Please Read All Instruction Before Using The Baby Scale, Especially These Few Fundamental Safety Instruction:

- Never leave your baby unattended while using your Baby Scale.
- The Scale is not satisfactory for babies of more than 20 Kgs. An overload can cause damage and risks.
- Always Put the Baby Scale on hard surface, flat and horizontal, keep away from source of heat such as stoves, radiators, etc.
- Never use the scale in a room where aerosol (sprays) are being used or in a room where oxygen is administered.
- Do not use the Scale in the proximity of humid zone, like the bathtub, the sink, etc.
- Do not submerge the scale in water or any other liquid.
- When the Scale is functioning, keep it away from electromagnetic waves, like microwave ovens, mobile phones and cordless networks, because the weighing would risk being erroneous.
- Do not attempt to repair the scale yourself.
- Never leave anything on the scale's platform when you are not using it.

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

2. TECHNICAL CHARACTERISTICS

Capacity : 0 to 20 Kg.
Graduation : 5 Grams
Function Tare
High Glow Seven Segment Red LED Display for clear vision
Function of modification of unit of weight (Kg./Lb.)
Indication of low battery LO-BAT
Inbuilt Battery : 6 V. 1.3 Amp. Rechargeable with 8 to 10 Hrs. Backup (If Battery Full Charged)
Equipped with high precision sensor (Load Cell)
Net Wt. of the Scale Approx. : 3.700 GM
Environment of Storage : from 10°C to 55°C
Environment of Functioning : from 10°C to 40°C
Precision of Weight Measures : 0.2kg to 20kg (± 0.005 Kg.)

3. KEYS FUNCTION

$\frac{\text{On}}{\text{Off}}$: the on/off key turns the scale on and off.

Zero : Empty Weight & Deduction of Initial Weight
MR : Change the unit of weight Kg. to Lb.
MR, M+,H+ : For external Callibration (If required)

4. NORMAL WEIGHING FUNCTION

1. Push the button on to turn on the scale four bars are displayed
2. 0.000 is displayed after few second it will be show (0)
3. Put the baby on the Scale. The weight is set when H appears.
4. OL is displayed if the scale is overloaded.

5. WEIGHING FUNCTION WITH A TOWEL

1. Push the button on to turn on the scale four bars are displayed
2. 0.000 is displayed after few second it will be show (0)
3. Put a towel on the scale. The weight of the towel will be displayed.
4. When H appears, push the button ZERO immediately.
5. 0.000 is displayed.
6. Put the naked baby on the scale. The baby's net weight is displayed when H appears.
7. Leave the towel on the scale. If you withdraw the towel from the scale, the screen will display a negative weight.

6. UNIT SELECTOR FUNCTION (Kg. to Lb.)

1. Push the button on to turn on the scale four bars are displayed.
2. After you put the baby on the scale, push the button MR the unit with convert Kg to lb.
3. The Kg. unit of weight is displayed when the scale is switched on again the next time.

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western[®] Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

7. CARE & ATTENTION

- Place your scale on a hard , flat surface will ensure the greatest accuracy and repeatability.
- Always weigh your baby on the same scale placed on same floor surface. Uneven floors may Affect the readings.
- Your scale round up or down to nearest 5 gm. If you weigh your baby twice and get two different readings , your baby weight lies between the two readings.
- Clean your scale with the damp cloth but do not allow water to get in side the scale . do not use any chemical cleaning agents.
- Do not allow your scale to become saturated with water as this can damage the electronics.
- Avoid overloading the scale.
- Treat your scale with care -it is a precision instrument , do not drop or jump or down to them.

8. LIMITED WARRANTEE

Mehtab Electronics Pvt. Ltd. warrantees that the product is free from any material or manufacturing defect for a period of the one year as from the date of purchase expect as stipulated here below. The warrantee on this Company product does not cover damage caused by inappropriate or misuse, accident attachment of any unauthorised accessory, any modification made to the product or any circumstance of any kind beyond the control of Company. Company cannot be held liable for indirect, consequential or special damage of any kind. All implicit warrantees of the aptitude of the product are limited to one years as from the initial date purchase. On reception Company will repair or replace your appliance as appropriate and will return it to you. This warrantee must be carried out by Company Service Center. Any maintenance of this product entrusted to anyone other than the Company Service Center voids this warrantee.

8. GROWTH CHART

Boys' Growth Standards : birth to 12 months*

Months	Length (Cm)	Weight (Kg.)	Head circumference (Cm)
0	46.3-53.4	2.5-4.3	32.1-36.9
1	51.1-58.4	3.4-5.7	35.1-39.5
2	54.7-62.2	4.4-7.0	36.9-41.3
3	57.6-65.3	5.1-7.9	38.3-42.7
4	60.0-67.8	5.6-8.6	39.4-43.9
5	61.9-69.9	6.1-9.2	40.3-44.8
6	63.6-71.6	6.4-9.7	41.0-45.6
7.	65.1-73.2	6.7-10.2	41.7-46.3
8.	66.5-74.7	7.0-10.5	42.2-46.9
9.	67.7-76.2	7.0-10.9	42.6-47.4
10.	69.0-77.6	7.5-11.2	43.0-47.8
11.	70.2-78.9	7.4-11.5	43.4-48.2
12.	71.3-80.2	7.8-11.8	43.6-48.5

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western[®] Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

Girls' Growth Standards : birth to 12 months*

Months	Length (Cm)	Weight (Kg.)	Head circumference (Cm)
0	45.6-52.7	2.4-4.2	31.7-36.1
1	50.0-57.4	3.2-5.4	34.3-38.8
2	53.2-60.9	4.0-6.5	36.3-40.5
3	55.8-63.8	4.6-7.4	37.2-41.9
4	58.0-66.2	5.1-8.1	38.2-43.0
5	59.9-68.2	5.5-8.7	39.0-43.9
6	61.5-70.0	5.8-9.2	39.7-44.6
7.	62.9-71.6	6.1-9.6	40.4-45.3
8.	64.3-73.2	6.3-10.0	40.9-45.9
9.	65.6-74.7	6.6-10.4	41.3-46.3
10.	66.8-76.1	6.8-10.7	41.7-46.8
11.	68.0-77.5	7.0-11.0	42.0-47.1
12.	69.2-78.9	7.1-11.3	42.3-47.5

WARRANTY CARD

ModelCapacity.....

Bill No.....Date.....Sr. No.....

Customer Name.....

Address.....

Tel. :

Digital Baby Scale carries warranty for a period of one year from.....
to.....against any manufacturing defects. The company's
responsibility under this warranty will be repaired the defective machine. However
consumables like Battery, Rubber & Plastic Parts are not covered under the
warranty.

In the event of consequential damage, defects resulting from accident, negligence,
mishandling, attempted repairs by any unauthorised person other than the
company's service engineer, the warranty will cease to exist.

* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

* Due to constant upgradation design, price and features are subject to change any time without prior notice.



Western[®] Surgical

www.westernsurgical.in

Email : westernsurgical@gmail.com

Ph. : +91-281-2232871 / 2243159

Date : 29-10-2016

