



# ELECTRONIC FARADIC STIMULATOR

## USER MANUAL



**Western Surgical**

[www.westernsurgical.in](http://www.westernsurgical.in) Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)

Email : [info@westernsurgical.in](mailto:info@westernsurgical.in) Ph. : +91-281-2232871 / 2243159

### HOW ELECTRONIC FARADIC WORKS :-

The electronic faradic is an electronic device for the natural treatment of tissue. It can produce proper stimulation to the "Biological Active Points" or "Low Resistance Points" underneath the epidermis of the human body. The flexible pads are used for stimulating the aforesaid point for 15-20 seconds with an electrical pulses generated by the faradic.

### WHERE TO USE :-

The electronic faradic can be used by anybody at any place. It is a household low / high frequency treatment device effective for aching found in shoulders, Numbness found in the peripheral nervous system with the effect of massage. It is also effective for the problem found in blood circulation, relief of tiredness, aching found in muscle and other illness.

### THE BENEFITS :-

The benefits of electronic faradic have been firmly established throughout the world in professional rooms, because of the sheer simplicity, they are more portable than a trolley load of medicines and needles, and the fact that extensive study is not required to control many simple conditions. In hospitals, nursing home, physiotherapy departments. In the animal world good results are being reported widely, from musculo ligamentous problems, skin soreness, stress control and improved performance. Most reports received of relief obtained have been for the more common condition such as arthritis, Headache (Including migraine), neck and low back pain, sinusitis, hay-fever, asthma, dysmenorrhea, premenstrual tension, stress insomnia, sporting and industrial injuries, constipation diarrhoea, hemorrhoids and irritable bladder, carpal tunnel, circulation problems eye problems, cough diabetes, digestive problems, facial neuralgia, bell palsy, fluid retention, gouthangover, hiatus hernia, hiccups, hyperactivity with withdrawal from tranquilizer addiction, jet lag

### THEORY

The electronic faradic is an electronic device which produce stimulation to the "biological active points" the treatment application example figure indicates such process should be repeated applied to the various point which need special treatment

### ATTENTION ON APPLICATION :-

no much danger would be caused by electronic faradic. However. Application on the area where the heart lies should be avoided

Those who have the following symptoms should refrain themselves from using the device

- \* Infants under three years of age
- \* Patients who have a heart ailment
- \* Those who are frequent with epilepsy
- \* Women who are pregnant.
- \* Those who have a skin disease
- \* Those who have allergic symptoms on application.

Instead of giving treatment, the objective of Electronic Faradic lies in giving relief to the pain which a person suffers temporarily and eliminating uneasiness. As for chronic disease, please consult your medical doctor for treatment

Power Source :- Both dry battery & AC 220v. 50 Cycle  
Rexine Model :- (use 950D Dry cells)  
Sheet Metal :- (Use 9V Battery) 6F-22

### TREATMENT METHOD :-

Intreatment application chart 20 categories are shown for treatment.

The dots in each treatment drawing are main points. The treatment time for each treatment point is approximately 15-20 seconds and the treatment time is approximately 10-15 minutes 1-2 times per day until the symptom is reduced or eliminated

For greater comfort, the areas of body where the pads are to be placed should be clean and free from talcum powder, grease, oils or ointments. Electronic Faradic is ideally used after bath or shower.

### PREPARING THE ELECTRODES (PADS)

- \* Make sure securely connecting the flexible pads to the cord.
- \* Wet the inner side of flexible pads with towel to make transmitting function better.
- \* Fasten the straps around the area to be treated.
- \* Position the pads under the straps, it is essential that the pads are held firmly against the skin

### USING THE ELECTRONIC FARADIC :-

- \* Plug the cord into main (For AC Model)
  - \* Insert the electrode cord plugs into the jack of the main unit
  - \* Switch on the AC or Battery switch
  - \* Switch On the intensity (Min- Mid - Max) switch in clockwise direction and stop the setting with appropriate stimulation
  - \* Turn the frequency adjustment switch - (HI-LO in sheet model) (Rotary switch in Super Deluxe Model) to adjust the frequency high or low
- As you start the unit frequency lamp blinks at the same frequency as the stimulation.

### TROUBLESHOOTING :-

If your Electronic Faradic fails consult your dealer. Do not attempt to dismantle the equipment. Before requesting servicing, please read this manual thoroughly and check the following

\* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.



**Western Surgical**

[www.westernsurgical.in](http://www.westernsurgical.in)










Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)

Ph. : +91-281-2232871 / 2243159

# FOR TREATMENT OF

- HIGH BLOOD PRESSURE • LOW BLOOD PRESSURE • INSOMNIA
- STRESS • ASTHAMA • SKIN PROBLEMS • NAUSEA / MOTION SICKNESS
- GENERAL TONIC • SEXUAL VITALITY • MENSTRUAL PAIN • BLOCKED NOSE
- TOOTHACHE • HEADACHE/MIGRAINE • EYE PROBLEMS • NECK PAIN
- SHOULDER PAIN • LOWER BACK PAIN • PAIN IN HANDS
- PAIN IN ARMS ELBOW • PAIN IN LEGS

## FEW EXAMPLES OF TREATMENT APPLICATIONS

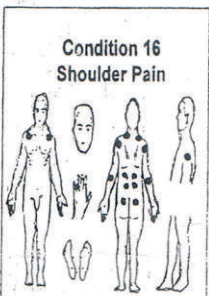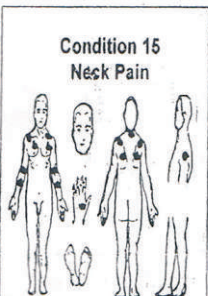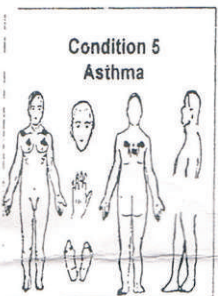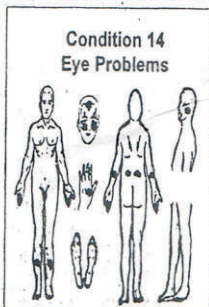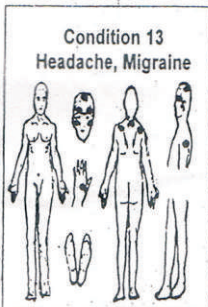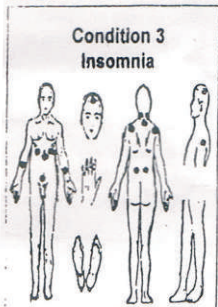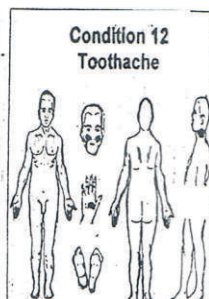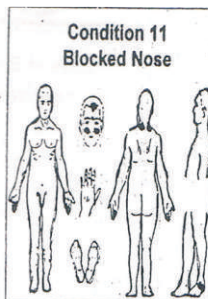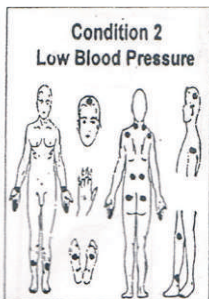

\* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.



**Western Surgical**

[www.westernsurgical.in](http://www.westernsurgical.in)


Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)

Ph. : +91-281-2232871 / 2243159




# FEW EXAMPLES OF TREATMENT APPLICATIONS


**Condition 7**  
Nausea/Motion Sickness

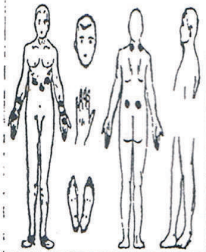


**Condition 8**  
General Tonic

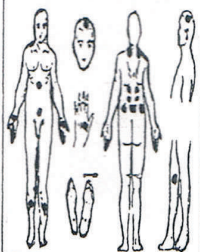


**Condition 17**  
Lower Back Pain

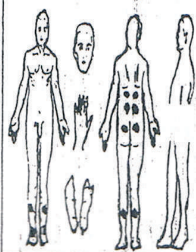


**Condition 18**  
Pain in Hands

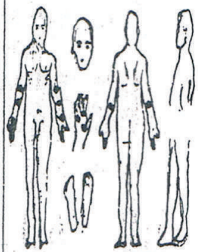


**Condition 9**  
Sexual Vitality

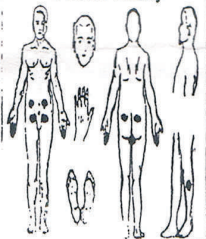


**Condition 10**  
Menstrual Pain

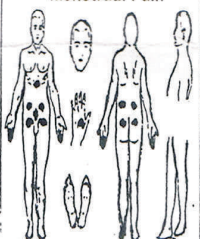

**Condition 19**  
Pain in Arms, Elbows

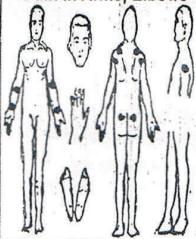

**Condition 20**  
Pain in Legs

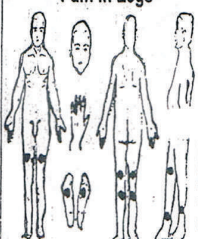

Symptom	Where to check action to take
Reduced stimulation or no Stimulation	<ol style="list-style-type: none"> <li>1. Is the battery exhausted ? → Replace the battery</li> <li>2. Is the Battery correctly connected ? Connect the battery carefully</li> <li>3. Is the Plug of main line &amp; Jack pins securely connected between cord and pads .</li> <li>4. Are the pads not clean or greasy ? → Clean them with wet towel</li> </ol>
	<p>Skin Reddens Prickly feeling</p> <ol style="list-style-type: none"> <li>1. Are the electrodes securely in contact with the skin ?</li> <li>2. Is the treatment time too long ?</li> </ol>
<p><i>Note</i> : Use only under the direction of your physician if attending a hospital or clinic .</p>	

\* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.



**Western Surgical**

[www.westernsurgical.in](http://www.westernsurgical.in)

Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)

Ph. : +91-281-2232871 / 2243159



### 7.3 WARRANTY CERTIFICATE

western surgical product is warranted to be free from electrical and mechanical defects in material and workmanship, under normal use. The western surgical product must not have been moved from the site of original installation. A new or remanufactured part to replace the defective part will be provided without charge for the part itself, through an authorized western surgical service dealer. The replacement part assumes the unused portion of the warranty.

#### WESTERN SURGICAL WILL NOT BE RESPONSIBLE FOR:

1. Damage or repairs required as a consequence of faulty installation or application by others.
2. Damage or repairs needed as a consequence of any misapplication, negligent handling, improper servicing, unauthorized alteration, or improper operations.
3. Failure to start due to voltage conditions, blown fuses, open circuit breakers or other damages due to the inadequacy or interruption of electrical service.
4. Damage as a result of floods, winds, fires, lightning, accidents, corrosive atmosphere, or other conditions beyond the control of western surgical
5. Parts not supplied or designated by western surgical.

<b>Date of Installation</b>	<b>Installed By</b>
<b>Model No.</b>	<b>Serial No.</b>
<b>Warranty Period</b>	
<b>Name of Doctor &amp; Address :</b>	

\_\_\_\_\_  
**Customer Sign. With Stamps**

\_\_\_\_\_  
**Marketed By: Western Surgical  
Sign. With Stamps**

#### **No Claim Warranty :**

- 1) **Any Defect Throughtout Power Supply**
- 2) **Any Physical Damage**

3) Under Warranty Standby Unit Not Provide

4) Under Warranty When Company Send Parts or Machine we Imidiat send to Buyer.

\* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.



**Western Surgical**

[www.westernsurgical.in](http://www.westernsurgical.in)

Email : [westernsurgical@gmail.com](mailto:westernsurgical@gmail.com)

Ph. : +91-281-2232871 / 2243159

Date : 18-5-2015

# OUR OTHER PRODUCTS RANGE



\* Carrying Warranty at Rajkot Office only on site warranty not provided free of cost.

\* Due to constant upgradation design, price and features are subject to change any time without prior notice.

## Western<sup>®</sup> Surgical

"Happy Home", 1st Floor, 23, Karanpara,  
B/h. S.T. Bus Stand, Kanak Road,  
Rajkot-360 001. Ph. : 0281-2232871/2243159  
Mo. : 98250 69069 - 98250 45997  
Email : westersurgical@gmail.com  
Email : info@westersurgical.in

\* [www.westersurgical.in](http://www.westersurgical.in) \*

